



## **ADVANCED SERIES BED ALARM MONITOR (w/ nurse call)**

**REORDER # 10260**

### **Instructions for Monitor Set-Up:**

1) Open the battery compartment and remove the protective film around the included battery and install the battery in the correct orientation as shown inside the monitor.

*NOTE: For safety, ensure that the batteries have sufficient power for proper monitoring. When battery is in use the front LED light will flash green every 5 seconds. When the battery is low, the front LED will flash every 5 seconds.*

2) Adjust the floor-pad toggle switch to your required usage of *floor* mat or chair/bed *sensor* pad. This setting changes the alarm function from a floor sensor monitor to a bed/chair sensor pad monitor.

3) Adjust the tone, volume and delay settings on the monitor to fit the user by using the appropriate controls inside the battery compartment.

*NOTE: If using the "Mute" option on the Bed Monitor, the monitor must be connected to the nurse call system using a cable for alarm annunciation. No local alarm will sound.*

4) Close battery compartment door. Secure with screw if desired.

5) Connect sensor pad to monitor by plugging modular plug into the appropriate jack on the monitor.

*NOTE: Verify that sensor pad is not expired prior to use.*

6) Place the sensor pad on the desired monitoring surface (bed or chair or floor) and attach monitor securely to a nearby area (such as a wheelchair back or headboard) using the included mounting bracket or strap.

*NOTE: Sensor Pads require sufficient pressure to compress pad when the user is positioned on it. If using over a cushion or other soft surface, test system before use to make sure the pad can be properly compressed.*

7) Place the patient on the pad surface to activate system and test to ensure function. (If placed properly; *in pad setting* the monitor will sound an alarm to notify staff if the user removes weight from the pad, and *in floor setting* the monitor will sound an alarm to notify staff if weight is applied to the pad.)

**WARNING:** The effectiveness of Monitors and Sensor Pad Systems depends on proper equipment installation and operation. Always test this system prior to use. Sensor Pad Systems are designed to help alert staff of situations that can lead to or result in falls, and are not a substitute for proper caregiver supervision. Please ensure that patients at high risk for falls are sufficiently monitored and help limit environmental hazards that can add to fall risk.

To reorder please call your local Proactive Medical Distributor.

**Toll Free:** 855-BE-PROACTIVE (855-237-7622)

**Call us:** 845-205-6004

**Fax us:** 845-205-6005

**Email us:** sales@proactivemedical.com